

# Penlon Medical Rail System

## SUCTION & OXYGEN THERAPY SOLUTIONS

- ◆ Strong aluminium construction
- ◆ Anodised extrusion
- ◆ Trolley mounted system also available
- ◆ Cost effective alternative to bedhead trunking
- ◆ Profile accommodates all commonly used rail clamp systems
- ◆ One, two or three metre lengths, (or custom lengths, up to three metres)



# Penlon Medical Rail System

SUCTION & OXYGEN THERAPY SOLUTIONS



## Penlon Medical Rail System comprises of aluminium rail, plastic insert strip and end caps and is suitable for all wall mounting and mobile trolley applications

- ◆ Strong aluminium construction able to support any device designed for rail mounting
- ◆ Complements Penlon's rail accessories range
- ◆ Profile accommodates all commonly used rail clamp systems
- ◆ Cost effective alternative to Bedhead trunking
- ◆ Trolley mounted system for resuscitation and ICU/CCU applications also available
- ◆ Mounts directly on the wall, using standard fixings

### The alternative to bedhead trunking

Medical Rail at the patient bedhead is a cost effective mounting system for medical equipment and the provision of patient therapy treatment

Flexible equipment placement - Services and equipment can be relocated easily.

Repair and maintenance - Instant access to equipment and services for repair and maintenance

### Rail Lengths

This versatile medical rail system is available in standard lengths (one, two or three metres), or in customer specified lengths (up to three metres)

NOTE Joining rods are required for rail runs over 3 metres in length

### Loading and Support

Recommended mounting centres	0.6 m
Maximum load	360 kg
Maximum torque	100 Nm

The rail loading must be reduced if it is necessary to position rail mounts significantly further apart than the recommended distance.

### NOTE

- 1 Installation must be carried out by a competent installer, who will supply screws and other fixings to suit site conditions
- 2 It is the responsibility of the customer to ensure that the supporting structure is adequate to take the expected imposed loads and to alter the mounting point centres if appropriate

### Rail Mounted Accessories

Increase functionality and performance with our range of rail mounted equipment and accessories

- 1 **Therapy Flowmeters**  
Rail mounted oxygen and air therapy flowmeters .
- 2 **Aneroid Sphygmomanometer**  
Rail mounted aneroid sphygmomanometers with velcro cuff
- 3 **Catheter Holder (Two or Five Bay)**  
2 or 5 bays box catheter holders (rail or V-plate mounting).
- 4 **Drip stand**  
Two hook drip stand.



Like us at [facebook.com/PenlonGlobal](https://www.facebook.com/PenlonGlobal)



Follow us at [twitter.com/PenlonGlobal](https://www.twitter.com/PenlonGlobal)

0473

Doc. No. MR0115SL / November 2015. Penlon and InterMed are trademarks of Penlon Limited. All other trademarks are the property of their respective owners.

Part of the InterMed Group © Penlon Ltd. 2015 All rights reserved

Penlon Limited  
Abingdon Science Park  
Barton Lane, Abingdon  
OX14 3NB, UK

**General**  
t +44 (0) 1235 547000  
f +44 (0) 1235 547041  
w [www.penlon.com](http://www.penlon.com)

**International Sales**  
t +44 (0) 1235 547001  
f +44 (0) 1235 547021  
e [international.sales@penlon.com](mailto:international.sales@penlon.com)

**UK Sales**  
t +44 (0) 1235 547036  
f +44 (0) 1235 547023  
e [uk.sales@penlon.com](mailto:uk.sales@penlon.com)

**Technical Support**  
t +44 (0) 1235 547060  
f +44 (0) 1235 547061  
e [tech.support@penlon.com](mailto:tech.support@penlon.com)

